MANUEL UTILISATEUR
USER'S MANUAL
MANUAL
GEBRAUCHSANWEISUNG
MANUALE UTENTE
GEBRUIKERSHANDLEIDING
KÄYTTÖOPAS
KASUTUSJUHEND
LIETOTĀJA ROKASGRĀMATA
دليل المستخدم

REF: 1907

PULSE® & DUAL PULSE®

COULEUR / COLOUR / اللون	SIMPLE PAVILLON / SINGLE HEAD / صفة اذن واحدة	DOUBLE PAVILLON / DUAL HEAD / صفة اذن مزدوجة
Stéthoscope Noir Black Negro Schwarz Nero Zwart Musta Must Juodas	507 500	507 400
Stéthoscope Azur Blue Azul Blau Blu Blauw Sininen Sinine Melyna	507 521	507 421
Stéthoscope Marine Navy blue Azul marino Marineblau Blu navy Marineblauw Tummansininen Tumssini Tamsiai mėlynas	507 522	507 422
Stéthoscope Rouge Red Rouge Rot Rosso Rood Punainen Punane Raudonas	507 530	507 430
Stéthoscope Gris altizé Grey Gris Grau Grigio Grijs Harmaa Hall Pilia	507 524	507 424
Stéthoscope Rose bougainvillier Rose Bougainvillea Bougainvillea rosa Rose bougainvillia Bougainvillea rosa Bougainvillea met rozen Ruoso Roos Różé	507 523	507 423
Stéthoscope Vert lagon Green Verde Grün Verde Groen Vireas Robetine Zalia	507 528	507 428

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USER'S MANUAL

Thank you for choosing the PULSE/DUAL PULSE stethoscope. Choosing Spengler's instrument is trusting the experience and recognized excellence since 1907. PULSE/DUAL PULSE is the result of numerous years of research and development aiming at providing the best quality design and comfort to health professionals.

This medical device is exclusively manufactured in accordance with the regulations in force, related to medical devices and is subject to quality controls.

INSTRUCTIONS FOR USE

Please read carefully the user manual before using the product and keep it safe for further reference.

INTENDED MEDICAL USE

PULSE/DUAL PULSE is a professional equipment for aid in the diagnosis. It is an acoustic instrument used for auscultation to listen to sounds within the human body in order to make a medical diagnosis: for example during heart, lungs and other organic noises auscultation as well as measuring the blood pressure.

INDICATIONS

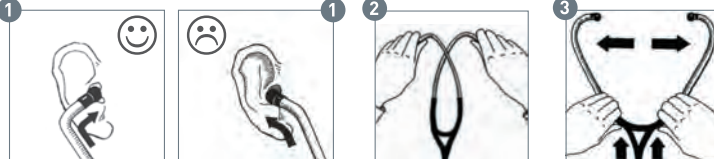
PULSE/DUAL PULSE stethoscope is a medical device Class I, for aid in the diagnosis, intended for health care professionals, trained for its use. It is used to listen to sounds within the body for gathering information about patient condition and to measure blood pressure for all types of patients: adults and children.

CONTRAINDICATIONS

- Do not used on injured skin.
- Do not used in case of instruments material allergies.

INSTRUCTIONS FOR USE

Fitting and adjusting the binaural tubes

The binaural tubes are designed to automatically adjust to your ears' listening angle. Ensure that the branches of the binaural tubes are pointing forwards, thus matching the direction of the auditory canal (fig. 1).

The binaural tube spring can be adjusted for improved comfort and better adjustment to your morphology.

To increase the spring tension, grasp the branches of the binaural tubes at the level of the earpieces and cross them over (fig. 2).

To reduce the spring tension, grasp the branches at the base of the binaurals and gradually move them apart to complete extension (fig. 3).

Chestpiece selection (double-sided model)

To select the desired chestpiece, gently tap this latter's membrane with the tip of your finger. If there is no sound, rotate the chestpiece through 180° until a click is heard.

Using the large chestpiece

The large chestpiece (diaphragm mode) is used to listen to highfrequency sounds.

Using the small bell chestpiece (double-sided model)

The small bell chestpiece is used to listen to low-frequency sounds.

Chest-piece	Single head	Dual Head
Material	Aluminium	Aluminium
Weight	22g	38g
Ø Diaphragm	44,5 mm	44,5 mm
Ø Bell	-	31,1 mm

Diaphragm (for large chestpiece)	Epoxy Resin Glass Fiber	Epoxy Resin Glass Fiber
Material	Epoxy Resin Glass Fiber	Epoxy Resin Glass Fiber
Diameter	44,1 mm	44,1 mm

Non-chill rings

Material	PVC	PVC
Ø Diaphragm	46 mm	46 mm
Ø Bell	-	34,2 mm

Binaurals and other parts

Binaurals length	63 cm	63 cm
Binaurals material	Aluminium	Aluminium
Spring material	Stainless steel	Stainless steel
Tubing material	PVC	PVC
Ear tips material	TPE	TPE
Total length	78 cm	78 cm

Accessories included

2 pairs of ear-tips, 1 diaphragm

WARNINGS

- Do not immerse the stethoscope in water/liquid.
- Do not use if the packaging is damaged.
- Do not sterilise.

SIDE EFFETS

None

MATERIOVIGILANCE

All serious incidents related to the use of a medical device should be notified to the manufacturer and the responsible authority of the Member State where the user and/or the patient live.

STORAGE AND HANDLING CONDITIONS

- Keep the stethoscope away from excessive heat or cold.
- Avoid exposing stethoscope to direct sunlight.

MAINTENANCE

Earpiece and diaphragm replacement

Earpieces:

Regularly ensure that the ends are properly clipped/screwed on to preserve good acoustic quality. To replace them, pull on the ends then replace them by pushing the new ends into place.

Diaphragm:

The diaphragm is held in place by the chill ring. To remove the diaphragm, grip the chill ring between thumb and index finger and pull to remove it from the chestpiece (fig. 4).

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UPKEEP AND CLEANING RECOMMENDATIONS

Regular cleaning of your stethoscope and in particular of the earpieces, ensures optimum acoustic performance.

- Clean the stethoscope with a cloth soaked in 70% isopropyl alcohol or moderately soapy water.
- Remove the earpieces before cleaning them thoroughly by soaking, then drying them. Ensure that no dirt remains inside.

DISPOSAL

Dispose of the device at the specific collection point according to local regulation.

QUARANTEE

The PULSE/DUAL PULSE stethoscope is guaranteed against manufacturing defects (parts and labour) under normal conditions of use, from the date of purchase (invoice date).

- Chestpiece, binaural tube assembly: 3 years
- Earpieces, diaphragm and chill rings: 1 year

The guarantee does not apply in the event of use non-compliant with instructions, improper use, modifications made by yourself or by an unauthorised third party, accidental damage and normal wear.

During the guarantee period, any stethoscope parts that present manufacturing defects shall be replaced and/or repaired free of charge, after returning the device to Spengler's after-sales service. In the event of a guarantee claim or of repairs, please return the complete stethoscope in its original packaging, accompanied by the dated invoice.

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