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Sommaire / Contents / الفهرس

Pièces détachées / spare parts / قطع الغيار	2
Etapes de montage / assembly steps / مراحل التركيب	3
Garantie / guarantee / الضمان	5

Importé et distribué par / Imported and Distributed by
مستورد وموزع بمعرفة

Holtex

30 rue Jean de Guiramand
13290 Aix-en-Provence
France

☎ +33 (0)4 42 90 31 31
✉ info@holtex.fr
🌐 www.holtex.fr

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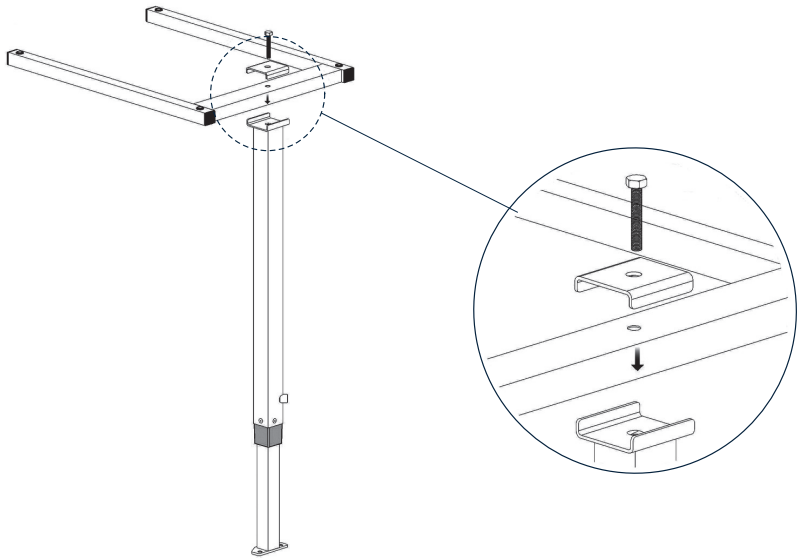
For after sales service, please download the form
from our website at :
www.gsh-med.fr

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Assemble the structure and the stainless steel rod by screwing down the mounting part.

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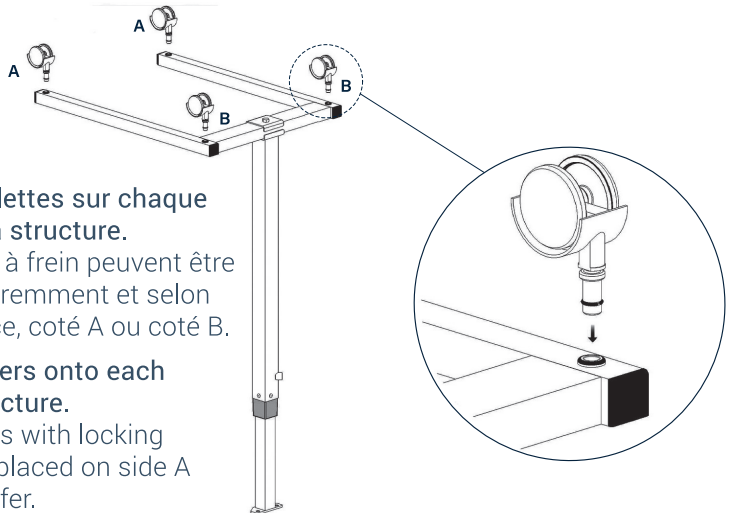
Insert the casters onto each end of the structure.

The two casters with locking brakes can be placed on side A or B as you prefer.

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
Screw the removable tray structure onto the stainless steel rod.

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

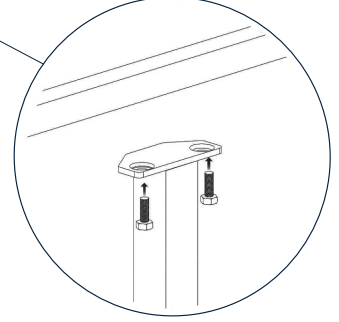
Place the tray onto its structure and insert the knob used to adjust the height to achieve the end result.

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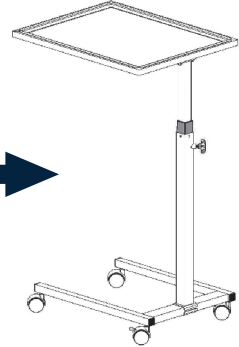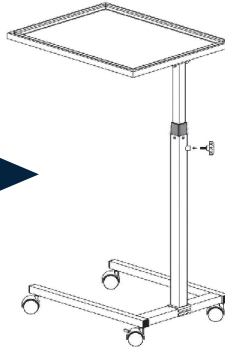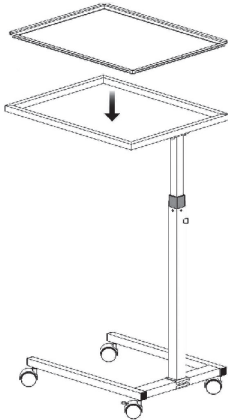EN Thank you for purchasing a Holtex product. Holtex furniture is guaranteed for one year and has been rigorously quality checked so that we can guarantee its replacement or repair should any proven defect appear during the guarantee period.

Holtex promises to replace defective parts returned (after a check is made by Holtex after-sales service) during the applicable guarantee period and as part of normal product use.

The guarantee does not cover:

- Wearing parts and consumables
- Travel and labour costs
- Failures that result from improper use, obvious negligence or transporting equipment
- Damage, breakdowns, failures or faults due to outside causes (natural disasters, electric power overvoltages, impacts, presence of foreign objects, etc.)
- Damage due to corrosion or gradual product deterioration
- Scratching, chipping or scuffing of the equipment that does not affect its proper operation
- Equipment where the serial number has been modified, made illegible or removed.

Repair or replacement under guarantee does not extend or renew the guarantee period.

The guarantee only applies if the complete product is returned along with the bill or original sales receipt issued in the name of the consumer by the retailer, who must have filled in the After-Sales form available for download from our website at www.gsh-med.fr

Holtex will ensure the availability of spare parts for products that it no longer sells for a period of two years after the end of the guarantee period.

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