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Cleaning, pre-Disinfection and disinfection

• The tubes(s) is(are) equipped with connector(s) and locking cap(s) or clamp(s) for cleaning and pre-disinfection.

• **Washing** (fastened cuff) : 40° C maximum machine wash.

• **Cleaning/pre-disinfection** (fastened cuff) : immerge in a suitable hospital pre-disinfectant (ex. in a solution diluted at 0.5%, ANIOSYME DD1 during 5 minutes or HEXANIOS G+R during 15 minutes) ; rinse in water and dry thoroughly.

• **Disinfection** (opened cuff) : spray IP sterile isopropolic alcohol (70 %) overall surface of the cuff - let dry.

Disinfection is possible with bleach solution, diluted as 0.5 % of active principle. Rinse with sterile water if needed.

When using pre-disinfectant or disinfectant solutions, always observe manufacturer's recommendations for concentrations and times.

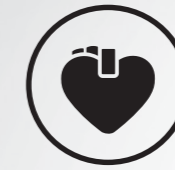
Characteristics

- Cuff is made of nylon with polurethane coating
- Tube is made of PVC (shore 50).

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Clinic® cuffs were designed to fight effectively against the nosocomial infections. They are used daily by thousands of professionals within the framework of septic isolations and the fight against epidemics.

WWW.SPENGLER.FR

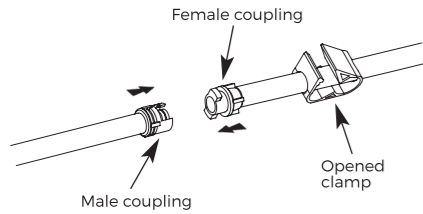
Connection for
tensiometer or monitor

CLINIC® CUFF


CUFF DECONTAMINABLE BY IMMERSION

• Connection to sphygmomanometers


1 - To turn and separate the connectors (male and female)

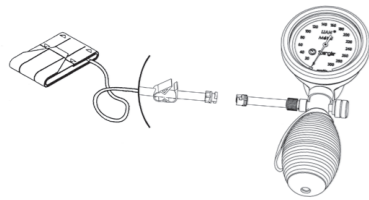2 - 1 outlet cuff: connect male coupling of the hand-held BPM to the cuff's female coupling

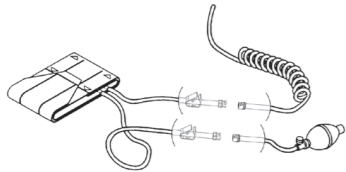3 - 2 outlets cuff: connect one of the cuff's female couplings to the coiled tube's male coupling. Connect the other cuff's female coupling to the bulb's tube male coupling.

• Cuff application

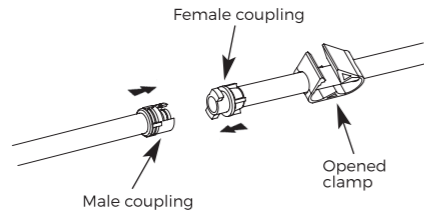
1 - Apply the middle of the bladder (arrow "ARTERE/ARTERY") on the artery and encircle the arm. The arrow is used as an index for the recommended range marking.

2 - Check the cuff size is correct for the arm circumference:


The index (arrow "ARTERE/ARTERY") should fall within the recommended range (in-between the arrows where shows the marking "ADULT/ADULTE"). Beyond these limits, the values are either overestimated (too small bladder) either underestimated (too large bladder).

• Cleaning, pre-disinfection and disinfection

1 - Disconnect the connectors (male and female).

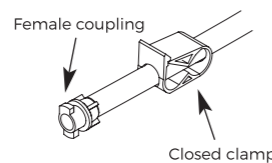2 - Obtainure the tube with the female connector with the clamp in order to keep the cuff watertight.

3 - Washing : in machine or by hand at 40°C maximum.

- Pre-disinfection : immerse in a cold solution (ex. ANIOSYME DD1 diluted at 0,5 % during 5 minutes) ; rinse thoroughly and dry flat.

- Disinfection : spray isopropyl alcohol (70 %) IP sterile ; dry flat.

Note : To take the connector out of the tube, cut the tube near the connector base and then cut it lengthwise with a blade.

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