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Plan Dur

Réf. SBRAN11_O

I. Descriptif :

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Zhangjiagang Xiehe Medical Apparatus & Instruments Co.,Ltd.
NO.7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

EC **REP**

SUNGO EUROPE B.V. :
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel
Netherlands

User Manual Spine Board

Réf. SBRAN11_O

I. Technical Features:

Product size (L x W x H): 184 x 45 x 6cm
 Packing size(1pcs/carton): 188 x 46 x 7cm
 N.W.: 7kg G.W.: 9kg
 Load bearing: less than 159kg

II. How to use:

This spine board with integrated safety belt is made from polyethylene. Highly resistant, it can also be used as a floating device and causes no interference with X-ray systems. It is also compatible with most head immobilizers and straps. It is mainly used in hospitals, sports facilities and ambulance services to transport patients or injured persons.

III. Cautions:

When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake. In general operation, be careful not to out torn the stretcher surface.

IV. Maintenance:

Keep clean as routine (incl. sterilization).
 Often check if any parts loose or not.


V. Storage & transportation:

Store this product in the place of damp proof & non-corrosion.
 Common transporting vehicle can carry this product.

	Manufacturer
EC REP	Authorised representant of the European Union
	Importer
	Distributor
REF	Holtex reference
LOT	Lot number
	Manufacturing date
MD	Medical device

Zhangjiagang XIEHE MEDICAL Apparatus & Instruments Co.,Ltd.
 NO.7th, Middle Xinzha Road, Zhashang Industrial Zone,
 Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China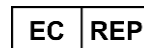

SUNGO EUROPE B.V. :
 Fascinatio Boulevard 522, Unit 1.7,
 2909VA Capelle aan den IJssel
 Netherlands