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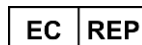
Zhangjiagang Xiehe Medical Apparatus & Instruments Co.,Ltd.
NO.7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V. :

Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel
Netherlands

User Manual

Foldable Stretcher in 2

Réf. SBRAN09_O

I. Technical Features:

Product size (L x W x H): 207 x 54.5 x 13cm

Folded size (L x W x H): 207 x 9 x 17cm

Packing size(1pc/carton): 211x 10 x 20cm

N.W.: 5kg G.W.: 6kg

Load bearing: less than 159kg

II. How to use:

This folding stretcher is made from high-strength aluminum alloy and waterproof, flame-retardant, anti-cracking fabric. Easy to transport and store, this outrigger stretcher can be folded lengthwise.

Approved for its efficiency, this stretcher is suitable for both ambulances and hospitals when transporting injured or sick people in emergency situations.

III. Cautions:

When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety's sake. In general operation, be careful not to out torn the stretcher surface.

IV. Maintenance:

Keep clean as routine (incl. sterilization). Clean with a surface disinfectant.

Often check if any parts loose or not.


V. Storage & transportation:

Store in a place protected from humidity, which could lead to product corrosion.

	Manufacturer
EC REP	Authorised representant of the European Union
	Importer
	Distributor
REF	Holtex reference
LOT	Lot number
	Manufacturing date
MD	Medical device

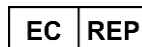Zhangjiagang Xiehe Medical Apparatus & Instruments Co.,Ltd.
 NO.7th, Middle Xinzha Road, Zhashang Industrial Zone,
 Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V. :

Fascinatio Boulevard 522, Unit 1.7,
 2909VA Capelle aan den IJssel
 Netherlands