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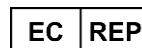
Zhangjiagang Xiehe Medical Apparatus & Instruments Co.,Ltd.
NO.7th, MiddleXinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel
Netherlands

User Manual Stair Stretcher

Réf. SBRAN14_O

I. Technical Parameters:

Product Size (L×W×H): 84×52×92cm
 Folding Size (L×W×H): 91×52×19cm
 Packing Size (L×W×H): 96×54×20cm
 N.W: 9kg G.W.: 11kg
 Loading Bearing: ≤159kg

II. How to use:

This evacuation and transfer chair features a high-strength aluminum alloy frame. It is ideal for emergency personnel who need to transfer patients up and down stairs. The two fixed rear wheels enable patients to be moved over most surfaces with minimum effort. The stretcher ensures patient safety during the transport process thanks to its 3-piece retaining straps with quick-release buckles. Under the front of the stretcher are two flexible lifting rods. The stretcher can be folded to save space in the ambulance car.

III. Cautions:

When carrying patients with this stretcher, be sure to lock tightly the fender locking device & fasten the safety belt for safety sake.
 In general operation, be careful not to torn out the stretcher surface.

IV. Maintenance:

Keep clean as routine (incl. sterilization).
 Often check if any parts loose or not.


V. Storage & transportation:

Store this product in the place of damp proof & non-corrosion.
 Common transporting vehicle can carry this product.

	Manufacturer
EC REP	Authorised representant of the European Union
	Importer
	Distributor
REF	Holtex reference
LOT	Lot number
	Manufacturing date
MD	Medical device

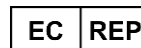Zhangjiagang Xiehe Medical Apparatus & Instruments Co.,Ltd.
 NO.7th, MiddleXinzha Road, Zhashang Industrial Zone,
 Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.
 Fascinatio Boulevard 522, Unit 1.7,
 2909VA Capelle aan den IJssel
 Netherlands