

MOBILE STAND FOR PNEUMATIC TOURNIQUET

TECHNICAL SHEET

Mobile stand for DDM pneumatic tourniquets
made in France !



1/3

The mobile stand is for use with the DDM pneumatic tourniquets.

It's delivered with :

- A mounting plate specially designed for DDM pneumatic tourniquets, equipped with a handle to facilitate movement,
- A five castors stand 2 with brake,
- A storage basket for the pneumatic armbands and extension cords.

REFERENCES


REFERENCE	DESIGNATION	DIMENSIONS IN CM		WEIGHT IN KG
		HEIGHT	FOOTPRINT	
A10701	Mobile stand for pneumatic tourniquet	98	Ø 65	+/- 15kg

INSTRUCTIONS FOR USE

- The mobile stand is used with pneumatic tourniquets DDM.
- The pneumatic tourniquet is maintained on the mounting plate by screw-in fixing knobs.
- To prevent risk of device falling, do not propel the unit mounted on mobile stand, a handle is provided to make secured manoeuvre by pulling or pushing the device to cross ground obstacles.
- The moving is done by pushing forward. Keep one hand on the handle in case of height difference.
- To prevent inadvertent movement, it is strongly recommended to lock the wheels brakes.
- Any movement of the device must be performed while the unit is unplugged.
- Keep the electrical power cord away from the castors.

IDENTIFICATION AND CHARACTERISTICS OF DIFFERENTS SUBSETS

IDENTIFIER	DESIGNATION	CHARACTERISTICS
A	Fixing plate	Aluminium + epoxy paint
B	Gallows with 2 plates	Aluminium + epoxy paint
C	Basket	Powder-coated steel
D	Base with 5 branches	Powder-coated steel
E	Castors	2 brake castors + 3 single castors, polyamide wheel frame and body, polyurethane wheel rim, precision ball bearings

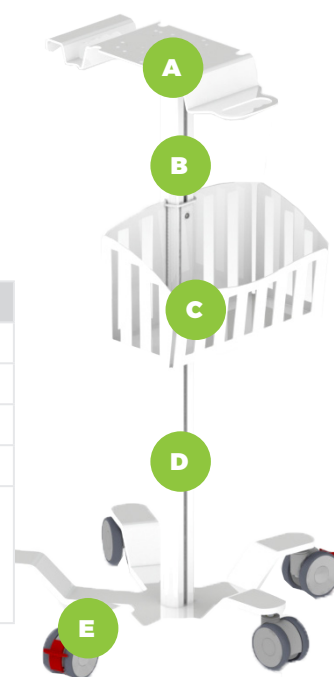

CLEANING AND DISINFECTION

Spray disinfectant solutions.

MAINTENANCE

Clean the wheels in order to remove residual dust or material which might interfere with the unit movement.

PACKAGING

REFERENCE	PACKAGING UNIT
A10701	1

STORAGE

TYPE OF PACKAGING	STORAGE AREA	TEMPERATURE	HYGROMETRY	ATMOSPHERIC PRESSURE
Original packaging	Ventilated area	-10° to 40° C	30 to 40 %	500 to 1060 hPa

GUARANTEE

The guarantee period is 2 (two) years from the date of purchase for normal usage conditions. The warranty do not cover poor or defective maintenance from the user. The product must be used in accordance with its intended use and may not be altered or subjected to misuse, abuse, accident or improper handling.

DDM FRENCH QUALITY

After-Sales Service

GSH provides mobile stands maintenance and repair services. Contact our Technical support center on: +33 (0)4 42 90 31 31.

Warranty

The guarantee period is 2 (two) years from the date of purchase for normal usage conditions. The warranty do not cover poor or defective maintenance from the user.