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**V800 PROJECTION VEIN FINDER VERTICAL  
SUPPORT INSTALLATION MANUAL**

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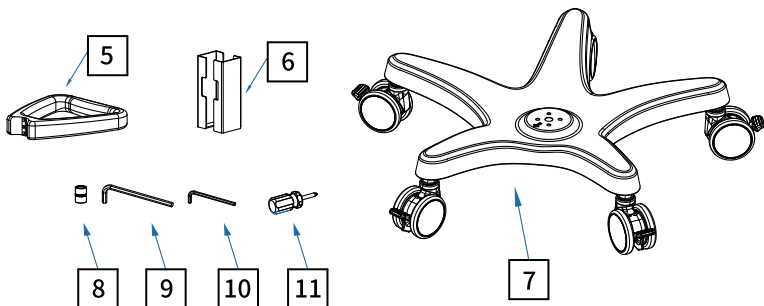
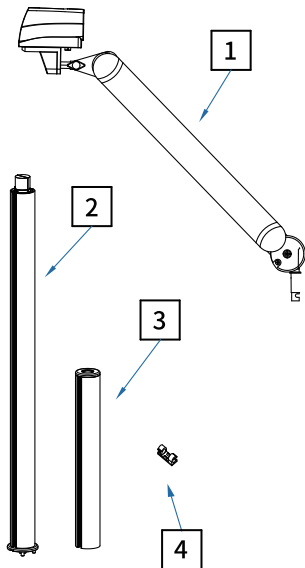
The following components will be used during this installation:

| N°<br>No | Description de la pièce<br>Component name                 | Quantité<br>Quantity |
|----------|-----------------------------------------------------------|----------------------|
| 1        | <b>Bras de support</b><br>Supporting arm                  | 1                    |
| 2        | <b>Colonne de support longue</b><br>Long stand column     | 1                    |
| 3        | <b>Colonne de support courte</b><br>Short stand column    | 1                    |
| 4        | <b>Passe-fils auto-adhésif</b><br>Self-adhesive wire clip | 3                    |
| 5        | <b>Poignée</b><br>Handle                                  | 1                    |
| 6        | <b>Boîtier adaptateur</b><br>Adapter box                  | 1                    |
| 7        | <b>Socle à roulettes</b><br>Base plate                    | 1                    |
| 8        | <b>Clé à douille</b><br>Sleeve-pipe wrench                | 1                    |
| 9        | <b>Clé Allen Carrée</b><br>Square wrench                  | 1                    |
| 10       | <b>Clé Allen 6 pans</b><br>Internal hexagonal wrench      | 1                    |
| 11       | <b>Tournevis cruciforme</b><br>Cross screwdriver          | 1                    |

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Sleeve-pipe wrench, Internal hexagonal wrench,  
Square wrench, Cross screwdriver.

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Install the long column and base plate

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
The diagram of base and stand column assembly position

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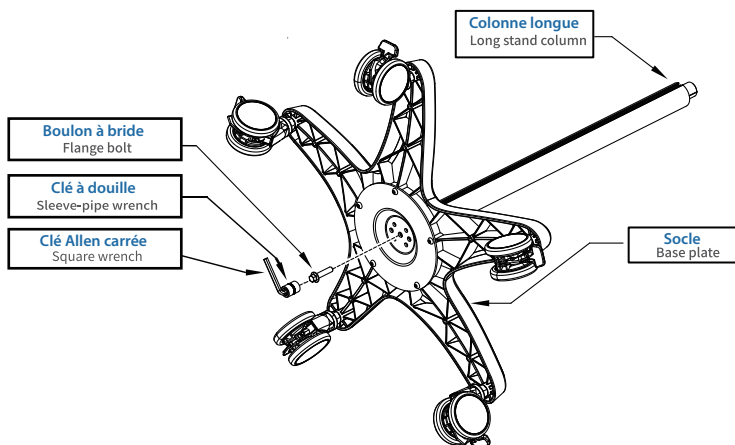Use the square wrench and sleeve-pipe wrench, loosen the flange bolts on the long column first. Attach the long columns to the base plate with the base plate arrow aligned with the column grooves. Connect the flange bolts to the long column through the base plate hole. Use square wrench and Sleeve-pipe wrench to tighten the flange bolts.

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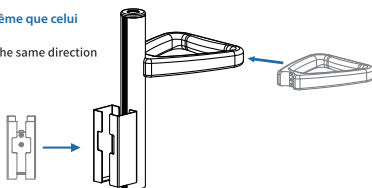
Loosen the inner hexagon screw while keeping the screw connection with the accessories, then slide the handle upward into the back slot of the column. Use the internal hexagonal wrench to tighten the socket screw clockwise.

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**Note:** The mounting direction of the handle should be in the same direction as the positioning bolt of the arm.

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Use the cross screwdriver to loosen the adapter screws, but keep the screws connected to the fitting, then slide the adapter box into the short column chute opposite the handle, and tighten the adapter screws with the cross screwdriver.

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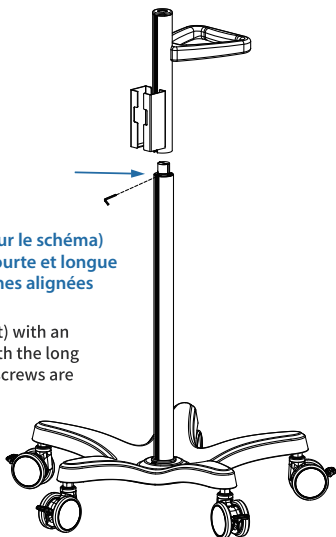
**Note:** the adapter box is not on the same side as the handle.

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Loosen the set screw (as in the arrow position on the right) with an inner hexagon wrench, then connect the short column with the long stand column and tighten the set screw. (note: when the screws are tightened, keep the two columns in-line)

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Unscrew the inner hexagon screw above the short stand column (note: do not discard).

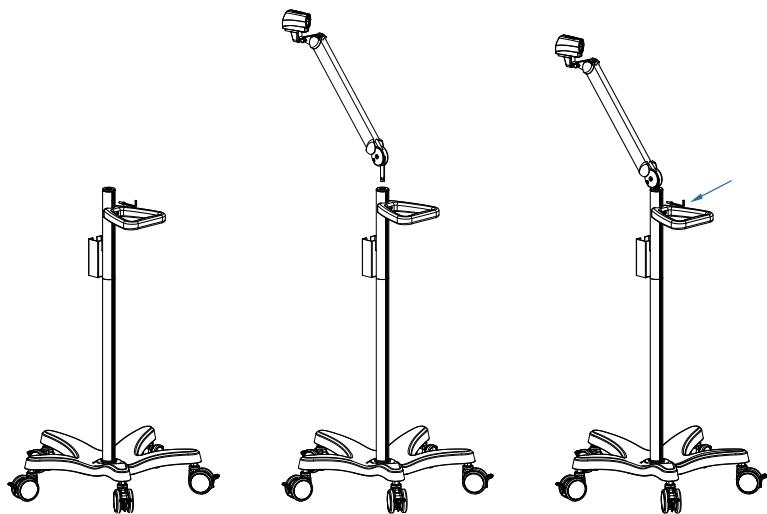
Insert the supporting arm into the short column.

Install the inner hexagon screw back to the short column and tighten it.

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Note: the supporting arm notch is aligned with the supporting arm positioning screw on the column.

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
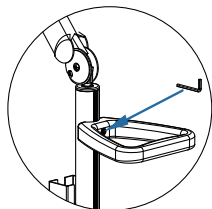
First, loosen the inner hexagon screw of the handle. Shake the handle when turning screw for 3 and a half turns to make it loose, move the handle up or down in balance and tighten the socket screw.

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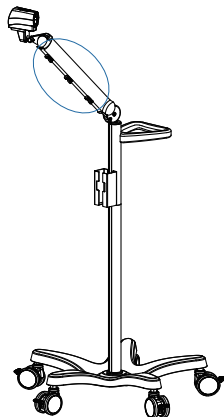Paste the self-adhesive wire clip (3) on the bottom of the supporting arm according to the diagram on the right.

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Insert the device into the supporting arm according to the arrow direction.