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ZHANGJIAGANG XIEHE Medical Apparatus & Instruments Co.,Ltd.
 NO.7th, Middle Xinzha Road, Sigang Village,
 Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China


SUNGO EUROPE B.V.
 Fascinatio Boulevard 522, Unit 1.7,
 2909VA Capelle aan den IJssel
 The Netherlands

User Manual

Emergency stretcher, foldable in 2, with headrest and wheels, blue

Réf. SBRAN07_B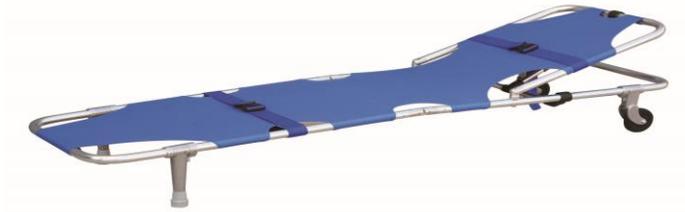

I. Technical parameters:

Product size (L x W x H): 185 x 50 x 24cm

Folding size (L x W x H): 92 x 50 x 11cm

Packing size (1pcs): 98 x 54 x 12cm

N.W.: 7kg G.W.: 9kg

Load bearing: ≤159kg

II. How to use:

This separable first-aid folding stretcher is mainly used for transporting patients with fractures. Both ends of the stretcher have hinge-type clutches to separate the stretcher into a right and left part. This allows for the patient to be quickly and gently placed onto the stretcher, operation table or bed.

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

III. Cautions:

When carrying patients with this stretcher, be sure lock tightly fasten the safety belt for safety sake.

In general operation, be careful not to out torn the stretcher surface.

IV. Maintenance:

Keep clean as routine (incl. sterilization).

Often check if any parts loose or not.

V. Storage & transportation:

Store this product in the place of damp proof & non-corrosion.

Common transporting vehicle can carry this product.

	Manufacturer
EC REP	Authorised representant of the European Union
	Importer
	Distributor
REF	Holtex reference
LOT	Lot number
	Manufacturing date
MD	Medical device

ZHANGJIAGANG XIEHE Medical Apparatus & Instruments Co.,Ltd.

NO.7th, Middle Xinzha Road, Sigang Village,

Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.

Fascinatio Boulevard 522, Unit 1.7,

2909VA Capelle aan den IJssel

The Netherlands