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Brancard pliable en 4 (alliage d'aluminium)

Réf. SBRAN08_O

I. Descriptif :

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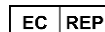
ZHANGJIAGANG XIEHE Medical Apparatus & Instruments Co.,Ltd.
NO.7th, Middle Xinzha Road, Sigang Village,
Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel
The Netherlands

User Manual

Folding stretcher in 4 (aluminum alloy)

Réf. SBRAN08_O

I. Technical parameters:

Product Size (L×W×H) : 208×55×13cm
Folding Size (L×W×H) : 104×17×9cm
Carton Size (2pcs) :: 108×18.5×18cm
N.W : 4.7kg G.W.: 11kg
Loading Bearing: ≤159kg

II. How to use:

The folding stretcher with outrigger is made of high strength aluminum alloy material and the waterproof leather materials. This stretcher with outrigger can be folded in length and width, easy for transportation and storage. The orange fabric is fire proof, water proof, anti-cracking and easy to clean.

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

III. Cautions:

When carrying patients with this stretcher, be sure lock tightly fasten the safety belt for safety sake.

In general operation, be careful not to out torn the stretcher surface.

IV. Maintenance:

Keep clean as routine (incl. sterilization).
Often check if any parts loose or not.


V. Storage & transportation:

Store this product in the place of damp proof & non-corrosion.
Common transporting vehicle can carry this product.

	Manufacturer
	Authorised representant of the European Union
	Importer
	Distributor
REF	Holtex reference
LOT	Lot number
	Manufacturing date
	Medical device

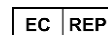ZHANGJIAGANG XIEHE Medical Apparatus & Instruments Co.,Ltd.
NO.7th, Middle Xinzha Road, Sigang Village,
Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel
The Netherlands