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PRECAUTIONS

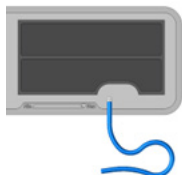
- The cuff should not be placed on a limb receiving another appliance (infusion, blood transfusion, SpO₂ sensor ...)
- Avoid tightening the cuff too tightly at the time of application, so as not to block the patient's blood circulation.
- During frequent blood pressure measurements, regularly check the condition of the patient's skin at the measurement site.
- Make sure there are no leaks in the cuff connector.
- Under no circumstances should a liquid penetrate inside the pneumatic part of the cuff, as this could partially or totally damage the electromedical device or manual measuring device to which the cuff are connected.
- For immersion cleaning of Easy Cuff® branded DDM and CLINIC®. In order to avoid any risk, we recommend DDM filling system with cap.

- Under no circumstances should a tubing be «clamped» using pliers or other unsuitable instruments.
- Under no circumstances should the cuff be handled with any sharp object.
- Under no circumstances should the cuff be inflated without being wound and closed with the two Velcro parts on a round limb or tube without a stop and sharp part.

Soft Cuff®; Easy Cuff® branded DDM and CLINIC® are not sterilizable.

CLEANING AND DECONTAMINATION

Dry cleaning (Soft Cuff®; Easy Cuff® branded DDM and CLINIC®)


Recommended product:

DETERQUAT / ANIOS.

- Spray a detergent, disinfectant, bactericidal foam on the cuff, taking care to distribute the product well.
- Leave on for 15 minutes.
- Rinsing is useless, wipe with a single-use wipe.

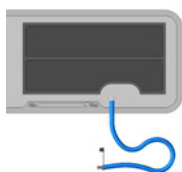Immersion cleaning (Easy Cuff® branded DDM and CLINIC®)

Recommended product:

ANIOS'CLEAN EXCEL D / STERANIOS.

- Seal the tube(s) with the closure system provided: Connector with cap.
- Immerse the tube and cuffs in a solution.
- Immersion time: 15 minutes.
- Rinse thoroughly with water, renewal of the soaking bath after each use.

GUARANTEE

SPENGLER guarantees that the cuffs comply with the specifications of the safety and performance standards applicable to it and currently in force. Cuffs can last for 10,000 cycles of use. All unused cuffs, kept in their original packaging and without damage are guaranteed for 12 months from the date of delivery.

All cuffs are warranted against design defects for the duration of their use.

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