

DATA EXPORT OPTION VIA ETHERNET FOR SIBELMED SIBELSOUND DUO A



Ref. DSI09505



18/08/2026 11:56:43 – For information purposes only, can be subject to change

DATA EXPORT OPTION VIA ETHERNET FOR SIBELMED SIBELSOUND DUO A



Ref. DSI09505

The way to export tests in various formats

Description

The Data Exporation option allows test results to be transferred to a network in three different formats (PDF, XML, or CSV) for later review and analysis.

To activate this option, a simple operation is to be done: by going to the "Maintenance" menu, then by clicking on the icon of a key. A screen will be displayed with 4 fields to enter the key, the serial number of the device and a numeric keypad. If the key is correct, a message will be displayed indicating the option you have enabled. Repeat the process if you want to enable the other option. Detailed description in the user manual and a video tutorial available.

Features

CE Marking	NONE	Medical device	No
Bar code	3661041009727		
DIMENSIONS AND WEIGHT			
Packed unit / Width (cm)	nc	Packed unit / Height (cm)	nc
Packed unit / Length (cm)	nc	Packed unit / Weight	nc
SPECIFIC STORAGE CONDITIONS			
Minimum temperature	NA	Max temperature	NA
Minimum humidity	NA	Maximum humidity	NA
OTHER FEATURES			
Packed unit / Quantity	1		